BIOPLAST® XTREME Deco

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To program the heating time in BIOSTAR® IV, please press the „heating“ key first and continue with pressing the key with the clock-symbol, enter the heating time in seconds and confirm by pressing the button with the clock-symbol once again.

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To program the heating time in BIOSTAR V / VII, please press the key with the clock-symbol, enter the heating time in seconds and confirm by pressing the button with the clock-symbol once again.

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Fabrication is identical for all BIOPLAST® XTREME Deco designs. In this instruction, the fabrication is shown by the example of "Lion".

Apply one thin layer of BIOPLAST® insulating agent only to those parts of the working model which will be covered by the mouth guard.

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Please embed the model as shown with the front side to the pressure chamber to avoid severe deformation of the decor due to stretching of the material

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Please embed the model in the pellets with the V-shaped part of the BIOPLAST® XTREME Deco covering the required extension of the anterior section (eg. From 3-3). Make sure the plate is fixed with the decor side down in SCHEU DENTAL machines. Heat the plate by setting the code (Scanner or keyboard) or recommended heating times according to the instructions. Lock pressure chamber and open after cooling phase has finished.

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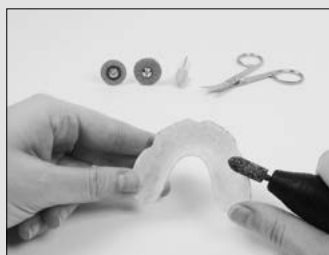
Hint: We recommend use of the "pellet cover" (REF 3006 / 3007) to avoid loss of pellets.

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Cut mouthguard on model bottom with scissors or scalpel, cover it with a moistened tissue and heat the occlusal surfaces under the infrared heater using the heating support. A light impression is done in the articulator with the insulated model of the antagonist. Cool with compressed air or water and remove mouthguard from the model.

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Cut mouth guard to the desired shape. Polish the edges with OSAMU polisher or DIMO®PRO wheel. Additionally, the edges can be finished thermally.