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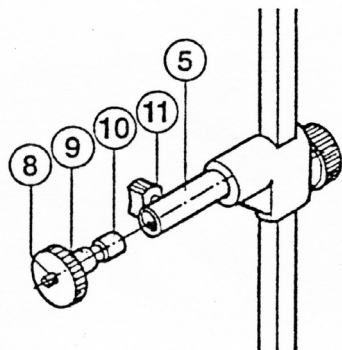
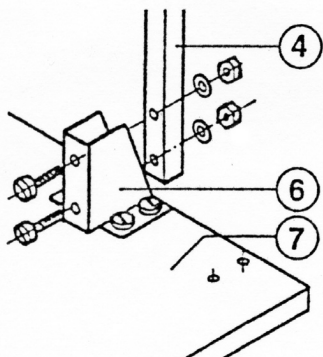
Operating Instructions CLEAR-ALIGNER Foto Tripod

Assembly

Take out of the box the column (4) with screws, the base plate (7) and the holding arm (5) and the midline indicator plate with glue-dots. Attach the column (4) to the socket (6) of the base plate (7) by means of the 2 screws and nuts.

Mounting the camera

Undo the locking screw (11) and take the bolt (10) with the connecting thread out of the holding arm (5). Please make sure the supporting plate (9) is tightly fixed on the bolt and screw it into the connecting thread of your camera (1/4"). Place the bolt with the mounted camera back into the holding arm. Adjust the camera horizontally, e.x. by means of the leveller of the model checker set and fix the locking screw (11).

Positioning of the camera

Undo the locking wheel (12) of the holding arm and shift the camera to the desired height, so that your Set-Up cast fills the view-finder.

Midline Indicator

For precise, reproducible positioning of the Set-Up cast fix the midline indicator plate on the base plate with the glue-dots, having it central and the angled part at the bottom of the view-finder.

By means of the angle and the midline, photos of the the Set-Up casts can be taken precisely, so that a 100% overlap (*Trans Window*) using the CLEAR-ALIGNER Software is reached.