# CA® Strip Opener

## Produktinformation

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## Product Information

### Description:

The CA® Strip Opener is a sterilisable, non-perforated instrument that is diamond-coated on both sides with serrated incisal edge.

### Indication:

The CA® Strip Opener is indicated for use in the IER technique, specifically for the exposure of an interdental gap in case of very close contact points.

### Intolerances:

Allergic reactions to the nickel content may occur in sensitised patients.

### Application/

In the first step, the interdental gap is opened using the CA® Strip Opener. In the next step, the gap is reduced to the required size by means of the CA® Stripping Tools according to the separation plan.

### Note:

The use of an oscillating EVA contra-angle handpiece is strongly recommended. The instrument is for single use only. To avoid jamming of the CA® Strip Opener, the instruments should be cooled with water.

### Safety note:

Improper handling may result in injury to the patient's mucosa/gingiva.

All information on processing our material - verbal, written or practical - is given to the best of our knowledge and must be read as references without obligation. Use of the instrument is beyond our control and lies within the responsibility of the user.

Product contains nickel.

For single use only.

Medical Device

CLINICAL PAPER 21:

### Delivery program:

CA® Strip Opener			
	Diamond particle size	Blade thickness	REF
Orange	15 µm	approx. 0.10-0.12 mm	5326
Red	40 µm	approx. 0.15-0.20 mm	5327

Manufactured by:  
OSUNG MIND CO.,LTD.  
57, Hwanggeum-ro 109 beon-gil,  
Yangchon-eup, Gimpo-si,  
Gyeonggi-do, Korea

EC/REP: EU-Representative:  
LITDENT UAB  
A. Mickevicius g. 2a, LT-76341, Siauliai, Lithuania  
www.oscident.de  
Amtsgericht Bad Homburg v.d.H., HRB 13152

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SCHEU-DENTAL GmbH