

# CLEAR-BLOKKER®

## Processing information

- Description:** CLEAR-BLOKKER® is a light-curing material for adjustments and build-ups in dentistry, orthodontics and dental sleep medicine.
- Indication:** CLEAR-BLOKKER® is a supplementary product to DURASPLINT® LC. It is intended for modelling canine guides and building up contact points for occlusal splints made of DURAN® 1.5 mm and DURASPLINT® LC. It can also be used for repairs of cracks and fractures of splint adjustments made of DURASPLINT® LC as well as for the fabrication of bracket transfer matrices in the lingual technique. The production of IST® splints is also possible using DURASPLINT® LC and CLEAR-BLOKKER®.
- Contraindication:** Any kind of intraoral application, e.g. for the fabrication of attachments or fillings as well as fabrication of frontal bite elevations ("jig") is not indicated. In addition, the building up of splints not being fabricated from DURAN® with a minimum thickness of 1.5 mm and DURASPLINT® LC is not indicated. The material is not suitable for these applications. In case of incorrect application, there is an increased risk of aspiration for the patient.
- Intolerances:** Allergies to individual components of the material may occur in sensitised patients. For details, please refer to the relevant safety data sheet at [www.scheu-dental.com/en/service](http://www.scheu-dental.com/en/service).
- Application/Curing:** Roughen the surface of the area to be treated on the DURASPLINT® LC adjustment, then clean and wet it using LC primer. The applied primer must be light-cured for 5 minutes in the LC-6 Light Oven. Apply CLEAR-BLOKKER® in a layer thickness of 1-2 mm directly from the dosing syringe using a cannula, before light-curing it for 10 minutes. The cured resin can be finished using a bur and then be polished. A UV LED lamp with a wavelength of 360-420 nm or the LC-6 Light Oven is required for polymerisation.
- Note:** CLEAR-BLOKKER® can also be used to fill smaller air bubbles in previously drilled or milled DURASPLINT® LC adjustments.
- Hazard statements:** Please observe the personal precautions for protective equipment and safe handling as indicated in the corresponding safety data sheet at [www.scheu-dental.com/en/service](http://www.scheu-dental.com/en/service).



### Delivery program:

CLEAR-BLOKKER®		
CLEAR-BLOKKER®	1 piece	REF 5261
4 × 1.5 g, incl. 10 cannulas		
Cannulas (Ø 0.4 mm)	5 pieces	REF 5277
Cannulas (Ø 0.9 mm)	5 pieces	REF 5278
DURASPLINT® LC, bar of 3, total approx. 40 g	1 piece	REF 3261
LC-Primer, 10 ml, with brush	1 piece	REF 3263
LC-6 Light Oven 230 V	1 piece	REF 5610
LC-6 Light Oven 115 V	1 piece	REF 5620

